

*Bob
AND
Brad*[®]



Eye Massager

User Manual

Mode: SD-300

Attention:

Please read and comprehend all the contents of this manual and operate the product according to the instructions.

DO NOT use this product if have eye diseases like Glaucoma, Cataract, and Retinal detachment. If eye ever carried out surgery, it is prohibited to use this product.

DO NOT clean the product with water, clean it with cotton soaked in alcohol.

DO NOT disassemble the product.

This product has a hot surface, people sensitive to heat should take extra care when using this.

This product is not intended to be used by people who are physically weak, slow in response, or mentally disabled, unless it is used safely under the guidance or help of the person responsible for their safety. Unattended young children and disabled persons are prohibited from using this appliance, and children are prohibited from playing with this appliance.

This product is intended for personal care use, it is not intended to replace medical care and medical devices.

DO NOT disassemble the product.

DO NOT impact or squeeze its interior lithium battery or expose it to fire.

DO NOT use when severe swelling occurs.

DO NOT use if the battery has been immersed in water.

Lithium battery set inside the product. **DO NOT** place in high temperature environment.

Consult your doctor if you have sensitive skin, allergic dermatitis, or orthopedic bio materials on face.

Remove contact lenses, hairpin and hair accessories before using.

Start the eye massager after putting it on, in order to avoid damage to the air bag due to over pressure.

DO NOT use in high temperatures or humid environments. Avoid direct exposure to the sun and gas corrosion.

This product will block the view. **DO NOT** use when walking or driving.

Only use the charger (5V/1A) to recharge the eye massager.


You must have your eyes closed while massaging.

Attention:

After massaging, the eyes are relatively blurred, and slowly become clear after closing the eyes for 1 minute.

DO NOT use sharp objects to scratch this product.

Product Structure:

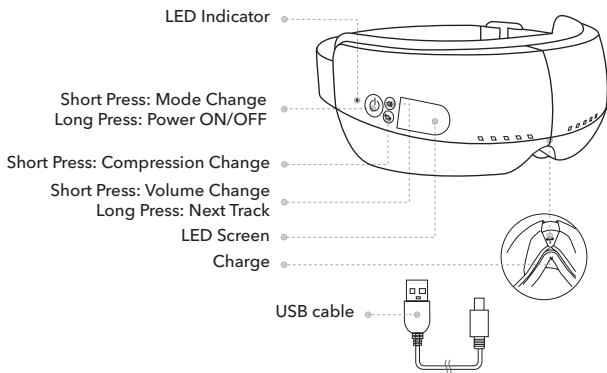

Technical Parameters:

AProduct Name: Eye Massager

Model: SD-300

Input: 5V $\overline{\text{---}}$ 1A

Rated voltage: 5V

Rated power: 5W

Color: White/Gray

What's included :

1x Eye Massager

1x USB Cable

1x User Manual


Wearing the device :

After massaging, the eyes are relatively blurred, and slowly become clear after closing the eyes for 1 minute.

Product Structure:

Align the cloth cover to your eyes, put on the device, adjust the elastic band, adjust the device to the most comfortable position.

Hold the power button for 3-5 seconds to turn on the eye massager.

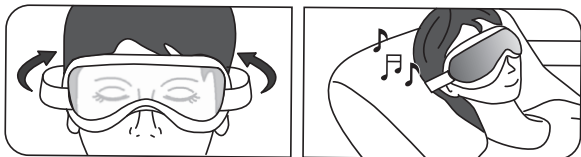

Attention

Remove any glasses, contact lenses, hairpins and hair accessories before use to avoid discomfort.

Do not operate the device with wet hands.

During operation:

Default Setting: Air Pressure + Heat + Music

Default Timer: 15 minutes

Press power button to change mode:


Hold power button 3-5 seconds to turn on/off the eye massager.

Gently press power button to change modes.


Audio voice will confirm mode change.




Key Control

Long press the "  " for startup and shut off.

Short press to change massager program, Mode 1 - Model 2 - Model 3 - Model 4 - Model 5.

Short press "  " to control volume, the volume will turn up/down continuously, Medium Volume - High Volume -Low Volume - No Volume.

Long press to change the music tracks, Beach Ambience with Seagulls -- Ambient Water Stream Running -- Ambient Piano -- Soft Rain

Press "  " to change air pressure program, Strong Pressure - No Pressure - Soft Pressure - Moderate Pressure

Bluetooth Music

Bluetooth pairing name: Eye Massager


Massager will play built in music by default, it will automatically switch to the Bluetooth device to play the music after mobile phone connection to the Bluetooth.

It will automatically restore the built in music play after Bluetooth disconnection.

Charging

1. Use the 5V mobile charger(not included) for charging your eye massager.
2. Insert USB cable into the socket of the mobile charger.
3. Softly insert the micro USB plug into the socket of the device,making sure the micro USB is inserted in the right direction.
4. Plug the charger into power.
5. While charging, the red LED indicator will show.When done charging, the LED will be green.The device will not work while charging is in progress.
6. The charge duration will be 2-3 hours.Disconnect the power adapter after charging is complete.

Charging

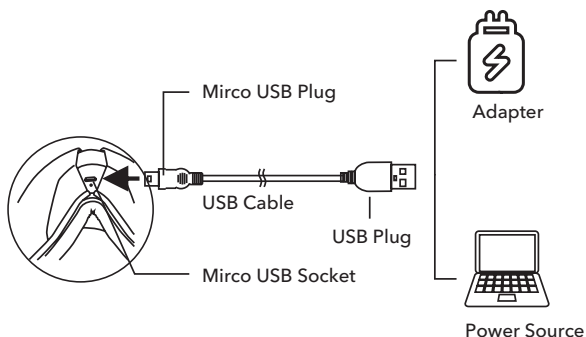

FAQ and Troubleshooting

Problem	Possible Causes	Actions
There is a sound mechanical or air sound coming from device	Normal sound of air sound pump inflating, motor working or air pressure	No need to do anything
Device turns off automatically. Device does not turn on. Buttons are not responsive. Air pressure is weak.	1. Battery is low. 2. The 15 minutes timer.	1. Charge the device. 2. Turn on the massager again.

Please cut off the power supply for the abnormality after being treated as above and then contact our service team. Do not disassemble the product for maintenance by yourself. Our company shall not be liable for product damages or personal injuries therefrom.

Cleaning and Maintenance

Attention

Please turn off the power when cleaning, do not use corrosive detergents such as benzene or diluents to clean the product.

Do not expose the device to excessive sun or hot temperature.

FCC Statement

This device complies with Part-15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference.
- (2) This device must accept any interference received, including interference that may cause undesired operation.

FCC Statement NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause any harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and the receiver.
- Plug the equipment into the outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help with the supplied fastener. **DO NOT** hang the unit by the power cord.

Correct Disposal of this product

This marking indicates that this product should not be disposed with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

A Dedicated Team Just For You



Every Bob and Brad product is backed by a 1-year warranty and an award-winning customer service team. If you are in any way dissatisfied with a product you ordered, we'll exchange it, replace it or refund your money under the warranty. Simply email us the problem, and we'll take care of it.



Troubleshoot

Help you solve any issues with your device.



Product Knowledge

Answer questions about your device and best practices.



Initiate Returns

Provide instructions for replacements, returns and refunds.



Place New Orders

Help with placing orders online and/or authorized resellers.

Get in Touch.

Call: +1 (612) 567-3035 Mon-Fri 9:00AM-4:30PM (CST)

Email: support@bobandbrad.com

Web: www.bobandbrad.com

Manufacturer (Made in China)

Shenzhen Laige Technology Co., Ltd.

C2-608, 6th Floor, Building C2, Bantian International Center,

Longgang District, Shenzhen, Guangdong Province, 518116

China

US Importer

FLIGE Products LLC

3001 Esperanza Crossing Austin, TX 78758

US

EU Importer

Linkfar Healthcare GmbH

Niederrheinstraße 71, 40474 Düsseldorf

Germany

UK Importer:

CET PRODUCT SERVICE LTD.

Chase Business Centre, 39-41 Chase Side, London N14 5BP

United Kingdom

